# BioSonic®

## UC125H (with Heat Feature)

### Ultrasonic Cleaning System

- ⓔⓃ Owner's Guide
- ⓓⓔ Benutzerhandbuch
- ⓕⓡ Guide d'utilisation
- ⓔⓢ Manual de usuario
- ⓔⓐ Guida per l'utente
- ⓓⓕ Gebruiksaanwijzing
- ⓈⓋ Bruksanvisning
- ⓓⓐ Betjeningsvejledning
- ⓕⓙ Käyttöohje
- ⓓⓞ Bruksanvisning
- ⓔⓐ Guia do Utilizador
- ⓔⓕ Εγχειρίδιο χρήσης
- ⓐⓗ 用户指南

## An Essential Part of Your Infection Control System

Congratulations on your purchase of a BioSonic® Ultrasonic Cleaner- the finest, most efficient and most popular ultrasonic cleaning system in dentistry.

Developed specifically for the dental industry, BioSonic units feature unique product design to optimize time and safety in the office or laboratory.

BioSonic is a total ultrasonic cleaning system, with a full line of cleaning solutions and unit accessories which provide you with the flexibility to efficiently clean instruments, prostheses and other appliances.

### What is Ultrasonic Cleaning?

Ultrasonic Cleaning is a process created by high frequency soundwaves. The soundwaves, enhanced by specially formulated cleaning solutions, create high energy cavitation. During cavitation, millions of tiny bubbles form and then collapse or "implode," releasing enormous amounts of energy and shock waves which scour the surface of instruments, appliances and other devices placed in the cleaning solution. This powerful scouring action reaches into minute crevices, which manual brushing cannot reach. The combination of energy and specially formulated solutions make ultrasonic cleaning the most effective method for removing gross and microscopic debris.

**Intended Use:** The Biosonic® UC125H Ultrasonic Cleaner is intended for cleaning medical instruments by the emission of high frequency soundwaves.

### I. Unpacking

1. Remove your BioSonic unit from the shipping container and inspect the unit for any possible damage which may have occurred during shipping.
2. Check the serial number of the unit (found on either the under side or rear of the unit) with the number on the shipping carton. If they are not identical, contact your dealer.
3. Fill out the warranty card and mail to Coltène/Whaledent Inc. within 10 days of receipt.
4. Select your desired location for the cleaner. It should be in close proximity to a sink or to waste lines to facilitate

draining. The hose supplied with the unit will allow you to place the unit on either side of the sink or connect directly to the waste line.

5. Connect one end of the hose to the drain valve (**Fig. 1**) and place the other end in the sink. If desired, have your plumber connect the hose directly to the waste line with the appropriate fittings.

### II. Setup Instructions

1. **CONNECT THE DISPLAY.** Attach the connector on the front of the display module to the connector on the back top right-hand corner of the unit. Using the supplied hex L-key tool, install two screws (included) to secure the Liquid Crystal Display (LCD).
2. **PLUG IN THE UNIT.** Attach the UC125H's electrical cord at the lower back of the unit and plug into a three-wire grounded AC outlet (in accordance with local regulations). It is recommended that the unit be plugged into a Ground Fault Circuit Interrupter (GFCI) outlet.
  - A. **SELECT A LANGUAGE.** Using the arrow keys on the LCD display, select the language code for your appropriate region. If you are unsure which code relates to your office, see the front page of the Owner's Guide and choose the language you use in your office. Use the arrow keys to highlight the appropriate code. (The code will flash when it is selected.) Press ENTER to accept.
3. **SET THE CLOCK.** Use the arrow keys to set the time on the UC125H. Four numbers will appear on the screen. Use the "+" arrow key to increase the digit; use the "-" arrow key to decrease the digit. Press ENTER to accept.

**NOTE:** Users who select English must choose AM or PM and press ENTER to continue.

4. **SET THE DATE.** Use the arrow keys to set the date on the UC125H. "01/01/00" will appear on the screen. Each number will flash when it is selected. Use the "+" arrow key to increase the digit; use the "-" arrow key to decrease the digit. Press ENTER to accept.

- US/Date Format (MM/DD/YY)
- International Date Format (DD/MM/YY)

5. **SETUP COMPLETE.** The display will now show the message "Setup Complete" in the language of your choice. Simply fill the tank of the UC125H with BioSonic Solution and water. You are ready to go! Enjoy your UC125H! The default screen will now display on the LCD screen, indicating that your unit is properly programmed.

**NOTE:** If a mistake is made while programming the LCD, a user may repeat this process by pressing and holding the "-" arrow key for 5 seconds.

**MEMORY FEATURE:** In the event of a power failure, your UC125H is equipped with a back-up power supply that will retain your programming information. This supply will be capable of storing user settings for a minimum of 12 days before it is again necessary to reprogram the unit.

### III. Filling Instructions

1. Insure that the blue drain handle (**Fig. 2**) is fully rotated clockwise to the "Close" position. Fill the unit with warm water to the fill line indicator (**Fig. 3**).
2. Pour the required amount of ultrasonic cleaning solution (either BioSonic General Purpose UC30, BioSonic Germicidal Cleaner UC38/UC42 or BioSonic Enzymatic Ultrasonic Cleaner UC32) and water into the tank. Press the solution reset key.
3. Change the solution in the tank daily, or sooner if it appears soiled or discolored. When beakers are used, change the solution in the beaker after each use.
4. Drain unit by fully rotating the blue drain handle counter-clockwise to the "Open" position and allow solution to empty into sink or waste system. Rinse tank and fully rotate the blue drain handle clockwise to the "Close" position. Never drain tank by manually discarding solution as this may cause the unit to stop working properly.

**NOTE:** Never operate the unit without solution. Operating a dry tank will void the warranty. Do not place objects directly on bottom of tank. Always use accessory baskets, cassette racks or beakers.

#### IV. Degassing Instructions

Degassing the solution eliminates large air bubbles that interfere with the cleaning process and is necessary each time new solution is added to the tank.

1. PRESS THE DEGAS KEY.
2. WAIT. The UC125H will begin to cycle on and off during the degassing process. The timer will count down from five minutes. The words “**Degas On**” will be displayed.
3. DEGAS COMPLETE! Once the timer reaches zero, the words “**Degas Complete**” will be displayed. Your solution is now ready for use. Pressing any key will take the user back to the main screen.

#### V. Operation Instructions

ACTIVATING THE UC125H. This section explains how to clean instruments with your BioSonic UC125H.

1. Press the ENTER key.
2. SELECT A LOAD. Three load icons will appear representing instrument baskets, beakers and, cassettes. To cycle through these icons:
  - a. Press the ENTER KEY to choose the appropriate icon for your particular load. The selected icon will flash; push ENTER KEY again to move to another icon.
  - b. If necessary, use the ARROW KEYS to increase or decrease the amount of cleaning time required per load icon (between 5 and 60 minutes). Press the ENTER KEY again to store the newly programmed default times.
  - c. Press the START/STOP KEY to begin cleaning.
3. WAIT. The UC125H will begin the cleaning process. The UC125H will continue to clean as the timer counts down the remaining minutes from the user-selected cleaning time. You may increase or decrease the cycle time during operation by pressing the ARROW KEYS up or down. Once the timer reaches zero, “Cycle Complete” will display on the LCD screen until any button is pressed. The LCD will then return to the main screen.

**NOTE:** The cleaning cycle can be stopped at any time by pressing the START/STOP

key. The unit will then return to the main screen.

Pushing the START/STOP KEY from the main LCD screen will start the last cycle time selected. To change the required cycle time, go back to step 2.

C-POWER: C-Power is an exclusive feature of the UC125H that will automatically adjust the power in the tank based on the load requirements. This function provides improved stability against varying solutions, solution level and instrument loads.

The BioSonic Ultrasonic cleaner should always be covered when operating to trap aerosols in the unit. The BioSonic UC125H is equipped with a “hinged” cover for easy one hand opening. The cover also conveniently lifts off for cleaning in spaces with low cabinets above the unit.

#### VI. Heater Operation Instructions

ACTIVATING THE HEAT FEATURE. The UC125H model is equipped with a heater to rapidly increase the temperature of the solution in the tank.

**NOTE:** The temperature in the tank is displayed in the lower right-hand corner of the LCD screen.

1. PRESS THE HEAT KEY. Pressing the HEAT key to activate this feature will bring up a new screen on the display. The first time this feature is activated, the user must choose to use Celsius or Fahrenheit. Using the Arrow Keys, select “C” for Celsius or “F” for Fahrenheit and press ENTER.
2. SET THE DESIRED TEMPERATURE. Use the Arrow Keys to increase or decrease the tank temperature between 25°C (75°F) up to 60°C (140°F) in 5° degree increments. Press START to accept this temperature. The unit will now begin to heat.
3. WAIT. The UC125H will now display the default screen and begin to heat to the desired temperature. The Heat Toggle will now display “ON.” Once the desired temperature is reached, the Heat Toggle will display “OK.” Current temperature information will always be displayed in the lower right-hand corner of the LCD display.

 <b>CAUTION!</b>	<p>The Heat feature of the UC125H is a pre-heat temperature only. Through prolonged use, the natural cavitation process will continue to heat the solution beyond the desired temperature. When this happens, the unit is still functioning properly.</p>
--------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

**NOTE:** Once the unit is programmed to use either Celsius or Fahrenheit, it will not prompt you to do this again. If a mistake is made or the user desires to change from one to the other option, after entering heat mode, press and hold the HEAT key for three seconds and then repeat steps 1 through 3.

#### VII. Solution Reset Operation Instructions

USING THE SOLUTION TIMER. The UC125H is equipped with a Solution Reset key. This feature tracks the amount of time your solution has been in use. The solution life is displayed in hours and minutes immediately above the SOLUTION RESET key. To utilize this feature:

1. PRESS THE SOLUTION RESET KEY once after every change of your ultrasonic cleaning solution. It is recommended to change your solution every 8 hours at a minimum, or whenever the solution becomes visibly cloudy or soiled. Coltène/Whaledent offers a variety of BioSonic ultrasonic cleaning solutions to meet your cleaning needs.
2. VERIFY. The solution timer should now display zeroes in the lower-left corner of the LCD display.

#### VIII. Cleaning Methods

##### A. In Tank

Basket Cleaning: Place parts to be cleaned into the basket (**Fig. 5, 6, 7**). Then place the basket directly into the main tank containing BioSonic General Purpose, Germicidal, or Enzymatic Cleaner. Clean according to instructions on bottle cleaner label.

##### B. In Beaker

For specialized item cleaning (e.g., prostheses): Place item into a beaker (**Fig. 8**), add desired cleaning solution (e.g., BioSonic Cement Remover or BioSonic Ammoniated General Purpose)

according to the products directions. Suspend the beaker in the tank by the use of the beaker holder and place the beaker positioning ring around the beaker so that the bottom 2/3 will be suspended in the main bath.

C. Instrument Draining

The UC125H is equipped with a bezel surrounding the tank. The bezel drains condensation back into the tank keeping the work area around the unit dry. The bezel also acts as a draining stand for the basket. Simply position the corner of the basket on two of the draining tabs (Fig. 4) at opposite corners of the tank and allow the solution to drain off instruments.

**IX. Accessory Use Directions**

A. Full Size FingerGuard™ BioSonic UC151XD Instrument Basket cleans large volumes of instruments and increases instrument processing efficiency.

Use Directions:

Place instruments across ridges in bottom of basket. Place basket in tank and clean for desired time. After cleaning rinse basket and instruments thoroughly under running water.

B. Three-Quarter FingerGuard: The UC152XD Basket provides you with the versatility of cleaning a basket load of instruments and beaker cleaning for prostheses (or other small items) simultaneously.

Use Directions:



Place Basket into UC125H (Fig. 5). Place instruments across ridges in bottom of basket. Place basket in tank and clean for desired time.

C. Beaker Holder

Place three UC153 Beaker Holders on the bezel. Place positioning rings around the beaker so that the bottom 2/3 of the beaker is suspended in the main bath. Clean for desired time.

**X. Environmental Conditions**

- Intended for indoor use.
- Maximum altitude 2000 m.
- 

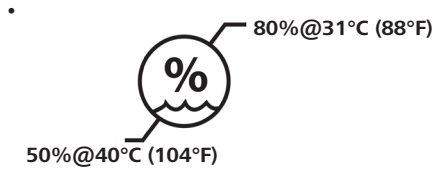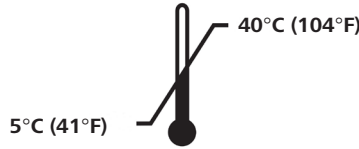- Main supply voltage fluctuations not to exceed ± 10% of the nominal voltage.
- Transient over voltages category II.
- Pollution degree 2.
- Class I equipment.

**XI. ⚠ Cautions and Warnings**

**NOTE: Failure to comply with these Caution and Warning statements may result in injury to personnel or equipment damage.**

 <b>CAUTION!</b>	Do not operate the unit without sufficient solution in the tank. Do not use water alone as the carrier bath. Water is not a satisfactory coupling agent unless a wetting agent is used. For best results use BioSonic General Purpose, BioSonic Germicidal Cleaner or BioSonic Enzymatic Ultrasonic Cleaning solutions.
	Do not place objects directly on bottom of the tank because they may block cavitation and cause buildup of heat in the transducer causing premature failure. Use only BioSonic UC125H accessory baskets, cassette racks or beakers to hold items.
	Do not restrict airflow. Provide adequate ventilation by maintaining at least 2 inches clearance at the back.
	Do not subject unit to shock or impact.

 <b>CAUTION!</b>	Do not immerse the unit in water.
	This product is intended for use only by trained healthcare professionals.
	Use this product in accordance with the instructions in the owner's guide.
	Unspecified or improper use of this product may impair safety protection.
	Before cleaning items of considerable value ultrasonically, consult the item's manufacturer for cleaning precautions.
	The power off selection from the keypad does not turn off main power to the unit.
	This equipment may cause radio interference or may disrupt the operation of nearby equipment. It may be necessary to take mitigation measures such as re-orienting or relocating the unit or shielding the location.
	Never drain tank by manually discarding solution as this may cause the unit to stop working properly.

 <b>WARNING!</b>	Use only the supplied power cord or equivalent.
	Do not disassemble the unit. There are no user serviceable parts inside.
	Keep unit covered to reduce airborne contaminants.
	Do not place fingers or hands in the ultrasonic tank while in operation.
	Operation for long periods of time will cause the cleaning solution to get very hot. To avoid scalding, do not place hands in the solution or spill any onto you. Be careful when removing an item from the tank as it may also be hot.

## XII. Order Information

Cat.No	Description	Qty
Unit and accessories:		
UC125H	100 Unit (100V, 50/60 Hz)	1
	115 Unit (†115V, 50/60 Hz)	1
	230CE Unit (230V, 50/60 Hz)	1
	230UK Unit (230V, 50/60 Hz)	1
	240 Unit (240V, 50/60 Hz)	1
UC155XD	Accessory Kit	
	3/4 FingerGuard Basket, (2) Beakers, (2) Covers, (2) Positioning Rings and (2) Beaker Holders	
UC151XD	Full Size FingerGuard Basket	1
UC152XD	3/4 FingerGuard Basket	1
UC153	One Beaker Holder	1
UC350	Replacement Drain Filters	3
UC360	Replacement Drain Hose	1
UC53	600 ml Beaker with Cover and Positioning Ring	1

### Tank Solutions:

UC30	General Purpose Cleaning Solution Concentrate in MeterDose® Dispensing Bottle (16 oz.)	473 ml
UC32	Enzymatic Ultrasonic Cleaner Concentrate in MeterDose Dispensing Bottle (8 oz.)	236 ml
UC34	Plaster & Stone Remover Powder Concentrate in Jar (1.85 lbs.)	840 g
UC38	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle (8 oz.)	236 ml
UC39	Cement Remover Concentrate in MeterDose Dispensing Bottle (16 oz.)	473 ml
UC42	Germicidal Ultrasonic Cleaning Concentrate in MeterDose Dispensing Bottle (16 oz.)	473 ml

## XIII. Maintenance

### A. Cleaning

All components should be cleaned each day by wiping with a soft cloth moistened with a commercially available mineral deposit remover followed by cleaning with isopropyl alcohol or a mild disinfectant.

#### CAUTION!

Do not use petroleum based solvents, iodophors or phenolic based products. (Iodophors and phenolics can stain the surface of the unit.) Clean up all liquid spills immediately. Rust deposits on tank can be cleaned/removed with a commercially available stainless steel cleaner.

### B. Stainless Steel Cleaning

Allowable:

- Wash with a clean cloth or rag soaked with warm water and mild soap or liquid dish detergent.
- Always clean in the direction of the original polish lines.
- Always rinse well with clear water (2 or 3 times) after cleaning. Wipe dry completely.
- Specialized household stainless steel cleaner may also be used.

Do Not Use:

- Steel or stainless steel wool or scrapers to remove stubborn debris.
- Harsh or abrasive cleaners.
- Do not let dirt accumulate.

Avoid:

- Cleaners that contain bleach.
- Products that contain chloride, fluoride, iodide, or bromide.

Chemicals harmful to use in the tank:

Acetophenone	Aluminum Chloride
Aluminum Fluoride	Aluminum Sulphate
Aluminum Bifluoride	Ammonium Chloride
Ammonium Hydroxide	Amyl Chloride
Antimony trichloride	Aqua Regia
Bromine	Calcium Bisulfate
Calcium Bisulfite	Calcium Hypochloride
Chloracetic Acid	Chloric Acid
Chlorine Anhydrous	Copper Chloride
Copper Fluoborate	Ethyl Chloride
Ferric Chloride	Ferrous Chloride
Ferric Sulfate	Fluoboric Acid
Flourine	Hydrobromic Acid
Hydrochloric Acid	Hydrocyanic Acid
Hydrofluoric Acid	Hydrofluosilicic Acid
Iodoform	Mercuric Chloride
Muriatic Acid	Phosphoric Acid
Sodium Hypochlorite	Potassium Chloride
Stannic Chloride	Stannous Chloride
Sulfuric Acid	Zinc Chloride

C. Inspecting the Power Cord Periodically check the power cord for damage and see that the connectors at both ends of the cord are fully seated.

### D. Changing the Fuse

1. Remove the power cord from the wall outlet first and then from the power cord receptacle on the rear of the unit.
2. The fuse compartment is located directly next to the power cord receptacle. To access, take a small, flat-headed screwdriver to the tab on the fuse holder. Apply slight pressure to release the fuse holder (**Fig. 9**).
3. Gently remove the fuse from the fuse holder.
4. Examine the fuse (**Fig. 10**). If blown, replace with a fuse that matches the unit's voltage and current (see section XIV). If fuse appears to be in good condition return the unit to your dealer or Coltène/Whaledent.
5. Plug the power cord into the power cord receptacle on the rear of the unit. Then plug the power cord into the wall receptacle.

## XIV. Unit Specifications

Transducers: 2

UC125H unit  
100 [100V 50/60 Hz, 137W (max.)]

115 [115V 50/60 Hz, 137W (max.)]

230CE [230V 50/60 Hz, 137W (max.)]

230UK [230V 50/60 Hz, 137W (max.)]

240 [240V 50/60 Hz, 137W (max.)]

Operating Frequency: 52 kHz

Tank Capacity: 1.25 gallon or 4.75 L

Use Capacity: 1 gallon or 3.78 L

Overall Dimensions: 16.5 x 10.5 x 13.8 in.  
42 X 27 X 36 cm

Tank Dimensions: 11.7 x 6.14 x 5 in.  
29.7 x 15.6 x 12.7 cm

Fuse Type: 5 x 20 mm Cartridge  
Time Lag: IEC 60127-2 Sheet 3

100-115V 2 Amp  
230-240V 1 Amp

EMC Class A: Suitable for use in all establishments other than domestic and those directly connected to a low-voltage power supply network which supplies buildings used for domestic purposes.

**XV. Symbols**

Caution, Hot Surface

Warning/Caution: Refer to accompanying documents

Caution: Risk of electric shock

Temperature Limits

Humidity Limits

Proper Disposal Required

**XVI. Warranty Information**

The UC125H has been developed specifically for use in dentistry and intended to be operated only by dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer or distributor, whether written, verbal or by demonstration, shall not relieve the dental professional of his/her obligation to control the product and to make all professional judgments regarding its use.

This product has been carefully manufactured and has met stringent quality assurance requirements. Our products are manufactured from new parts or new and serviceable used parts. Regardless, our warranty terms apply. If, in normal use, it is found defective in material or workmanship within a period of one year from date of purchase, it will be repaired at Coltène/Whaledent's option, or replaced. Tampering with any of the components, misuse, negligence,

alteration, water damage, accident or lack of reasonable or proper maintenance and care will void this warranty. Failure to use and/or maintain the product in accordance with the instructions contained in this manual will void this warranty.

**THE FOREGOING IS IN LIEU OF ANY EXPRESSED OR IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION, ANY WARRANTIES OR MERCHANTABILITY OR FITNESS FOR USE.**

**COLTÈNE/WHALEDENT ASSUMES NO RESPONSIBILITY FOR ANY INCONVENIENCE, LOSS, INJURY OR DIRECT, INDIRECT OR CONSEQUENTIAL DAMAGE ARISING FROM THE POSSESSION OR USE OF THIS PRODUCT.**

Any claim for damage or breakage in transit should be made at once against the carrier. If factory service is required, be certain to properly pack your equipment and return prepaid and insured to the factory.

**XVII. Disposal**

The Biosonic® UC 125 is an electric appliance requiring proper disposal according to local laws and regulations.

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© 2015
Coltène/Whaledent GmbH + Co. KG