# Ena White Gel Desensibilizzante

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### Descrizione:

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## Instructions for Use

### Description:

**Ena White Desensitizer Gel** is a water-based paste for remineralization and desensitization after the bleaching process, with a pleasant peppermint flavor.

### Directions for use:

Professional application, immediately before or following bleaching in dental office.

#### I. Custom tray application:

1. Preceding or following bleach application, rinse the custom tray thoroughly under running water.
2. Extrude a generous layer of **Ena White Desensitizer Gel** into the tray and apply to the upper and/or lower teeth.
3. Leave the tray in the mouth for a minimum of 3 minutes.
4. Remove the tray.
5. Ask the patient to rinse any **Ena White Desensitizer Gel** remaining on the surface. Any residual **Ena White Desensitizer Gel** can be left to gradually dissipate.
6. Any residual Ena White Desensitizer Gel in the tray should be rinsed or brushed off under running water immediately.
7. Advise the patient not to eat or drink for 30 minutes after the end of the treatment.

#### II. Non tray application:

1. If necessary, remove any excess saliva on the tooth surface with a cotton roll or pellet.
2. Apply a generous layer of **Ena White Desensitizer Gel** to the tooth surfaces using an application swab, gloved finger or difficult interproximal areas using a Tooth Cleaning Brush.
3. Leave **Ena White Desensitizer Gel** in the mouth for a minimum of 3 minutes.
4. Ask the patient to rinse any **Ena White Desensitizer Gel** remaining on the surface. Any residual **Ena White Desensitizer Gel** can be left to gradually dissipate.
5. Advise the patient not to eat or drink for 30 minutes after the end of the treatment.

### Side-effects:

With proper use of this medical device, un-wanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects — even if it is doubtful that the side-effect has been caused by our product — please kindly contact us.

### Contra-Indications / interactions:

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

### Storage:

Store in a dry low humidity area at 3-25°C away from direct sunlight.

Packages: Syringe 2 ml

MICERIAM S.p.A. - Via G. Marconi 83 - 16036 - Avegno (GE) Italy - [www.micerium.it](http://www.micerium.it)  
Fax: +39-0185-7887970 - Tel. +39-0185-77887870 - e-mail: [hfo@micerium.it](mailto:hfo@micerium.it)